INSTRUCTIONS FOR USE

*SomnoDent* HERBST ADVANCE ELITE<sup>®</sup>

*SomnoDent*<sup>®</sup> HERBST ADVANCE<sup>®</sup>

RxOnly

English  
Français  
Español


# Thank you for choosing a SomnoMed device

These instructions provide you with important information on the correct use and care of SomnoDent Herbst Advance and SomnoDent Herbst Advance Elite (hereafter referred to as "device").

**Important:** Please read this entire guide carefully before using your device. Contact your dental sleep professional should you have any questions.


## SomnoDent Herbst Advance Elite

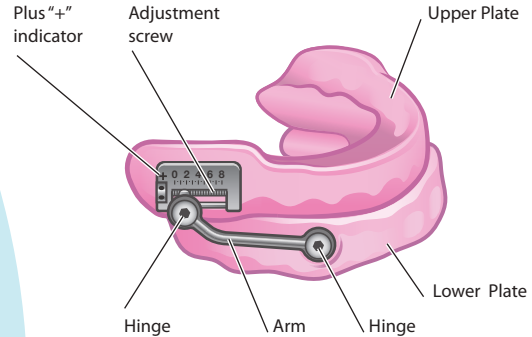## Storage Container

## Cleaning Brush

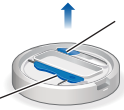## Adjustment Tool

## SomnoDent Herbst Advance

## Adjustment Key

## Release Tool

## Adjustment Tool

## Indications for use

---

The SomnoDent Herbst Advance and SomnoDent Herbst Advance Elite are intended for the treatment of nighttime snoring and mild to moderate obstructive sleep apnea in patients 18 years of age or older.

## Contraindications

---

These devices are contraindicated for patients who:

- » Have central sleep apnea
- » Have severe respiratory disorders
- » Have temporomandibular joint dysfunction (TMJ)
- » Have loose teeth
- » Have advanced periodontal disease
- » Are under 18 years of age

## Precautions

---

Dentists should consider the medical history of the patient, including history of asthma, breathing, or respiratory disorders, or other relevant health problems, and refer the patient to the appropriate healthcare provider before prescribing the device.

## Warnings

---

Use of these devices may cause:

- » Tooth movement and changes in occlusion
- » TMJ pain or tenderness
- » Obstruction of oral breathing
- » Excessive salivation
- » Gingival or dental soreness

## Before first use

---

Before first use, thoroughly clean your device by following the cleaning instructions in this manual.

## Possible side effects

---

You may experience some temporary side effects such as sore jaw joints, increased salivation, dry mouth, sore teeth, irritation of the soft tissue in the mouth and/or a slight change in bite.

These possible side effects should diminish within an hour of removing your device. If any of these side effects continue, contact your dental sleep professional and maintain regular sleep dental check-ups.


Changes in bite or jaw stiffness and pain upon removal of your device may be alleviated with clenching exercises. To assist with these exercises, use a SomnoMed Morning Repositioner or chew gum upon removal of your device. The Morning Repositioner is supplied separately. Please contact your dental sleep professional for more information.

## Using your device


### Important

- » Keep your device away from pets and small children
- » Always follow maintenance instructions for cleaning and care of your device
- » You may initially experience a mild plastic smell or taste. This will disappear with use
- » If any device components are damaged or broken, discontinue use to avoid injuries such as choking or laceration

## Fitting your device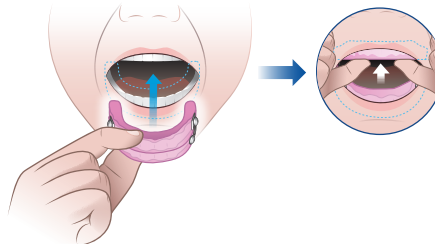


- 1** Hold the upper and lower plates together and mount upper plate to your top row of teeth first. Push upwards on the upper plate with both thumbs to ensure that it fits securely and comfortably.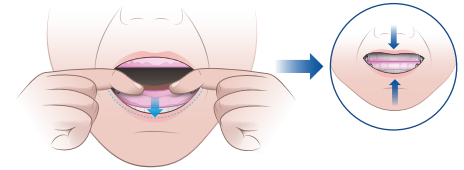


- 2** Mount the lower plate to your bottom row of teeth. Press down on both sides of the plate using your index fingers to ensure that it fits securely.
- 3** Once the upper and lower plates are securely in position. Slowly close your mouth until your lips lightly touch each other.

## Removing your device

---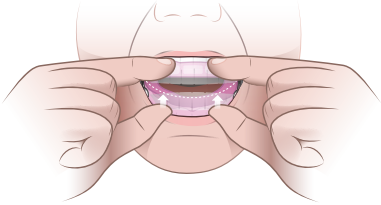

- 1 Remove the lower plate from the teeth first. Use your thumbs on both sides to push the plate upwards.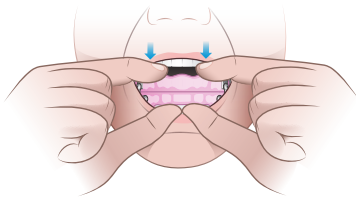

- 2 Gently pull the upper plate down using both index fingers and thumbs. Remove the device from your mouth.


## Adjusting your device

Use the adjustment screw to change the device's mandibular advancement between -1.0 mm to +7.0 mm from the starting position determined by your dental sleep professional.

Each 90° turn of the adjustment screw changes the mandibular advancement in approximately 0.09 mm steps.

Please refer to the section [“Using the adjustment screw”](#) to learn how to adjust using the screw.

**Important:** Your dental sleep professional will advise the amount of adjustment for your therapy.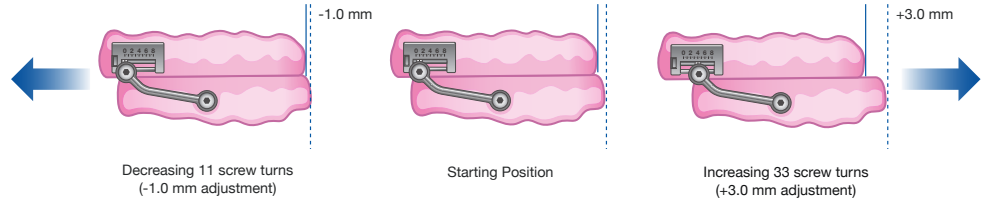


90° Turns	-11	1	11	22	33	44	55	66	77
Adjustment (mm)	-1.0	0.09	1.0	2.0	3.0	4.0	5.0	6.0	7.0

Decreasing ← Starting Position → Increasing


## Using the adjustment screw / Increasing mandibular advancement

### STEP 1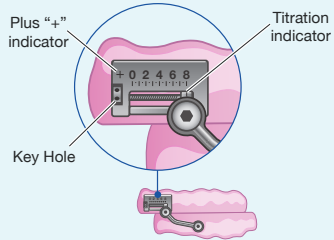


Hold the device with the titration indicator facing you and Plus "+" indicator at the top. The starting position set by your dental sleep professional is the 1.0mm position on the titration indicator.

### STEP 2

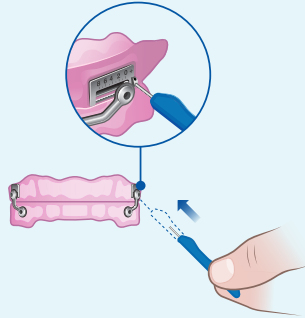
Locate the two keyholes and insert the adjustment key into the bottom keyhole.

### STEP 3

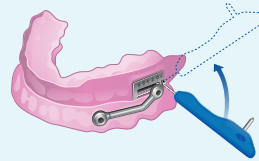Rotate the adjustment tool 90° upwards towards the Plus "+" indicator. Counting the number of turns, repeat until the desired adjustment is achieved on the titration indicator.


### STEP 4

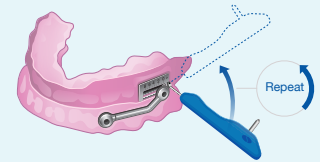**Important**  
Repeat on the other side, completing the same number of turns.


## Using the adjustment screw / Decreasing mandibular advancement

### STEP 1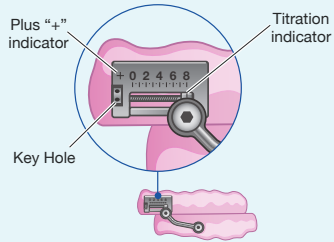


Hold the device with the titration indicator facing you and Plus "+" indicator at the top. The starting position set by your dental sleep professional is the 1.0mm position on the titration indicator.

### STEP 2

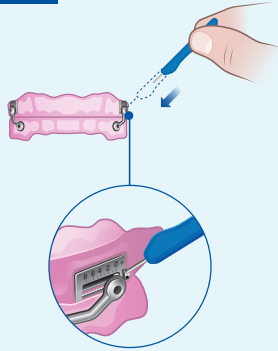
Locate the two keyholes and insert the adjustment key into the top keyhole.

### STEP 3

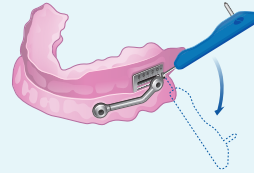Rotate the adjustment tool 90° downwards away from the Plus "+" indicator. Counting the number of turns, repeat until the desired adjustment is achieved on the titration indicator.

### STEP 4

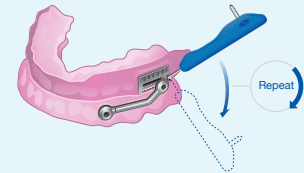
**Important**  
Repeat on the other side, completing the same number of turns.

## Cleaning

Clean your device immediately after each use

- 1 Rinse in cold or lukewarm water with a temperature no greater than 45°C/113°F.
- 2 Clean with a soft toothbrush and mild liquid soap.
- 3 Use the supplied cleaning brush to clean the area around the adjustment screw and hinges.
- 4 Rinse well in cold or lukewarm water.

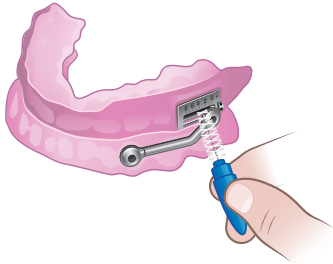SomnoMed recommends using SomTabs, cleaning tablets specially formulated to clean oral devices. SomTabs clean hard to reach areas to effectively eliminate odors.

## Caution

- » Never use hot or boiling water on your device
- » Do not use toothpaste as it contains abrasives that may damage your device
- » Do not use mouthwash to clean your device
- » Do not use ultrasonic cleaning
- » Do not use organic solvents like ethanol, acetone, etc

## Storage

Store your device in the storage container supplied with sufficient water to cover the device. Store your device out of direct sunlight.

## Traveling

Protect your device when traveling by using the supplied storage container. Ensure your device is thoroughly clean and dry before storing.

**Important:** Do not travel with your device stored in water or any other liquid.

## Maintaining the appearance of your device

---

To reduce the possibility of discoloration, always brush your teeth thoroughly before use. Do not smoke or drink coffee, tea, colas or wine with your device in place.

Amalgam fillings or the use of Pyralvex® (a solution for mouth ulcers and denture irritation) may cause discoloration of your device over time.

Discoloration of the material will not adversely affect the performance or life of your device.

\*Pyralvex is a trademark and brand of Meda AB, Sweden.

## Regular Inspection

---

Regularly inspect your device for signs of damage, including cracking. If your device is damaged or broken, it must no longer be used. This will avoid injuries, such as choking or laceration.

If your device is damaged, immediately contact your dental sleep professional. They will arrange for your device to be repaired by an authorized SomnoMed facility.

## Warranty

---

During the warranty period for material defects applicable in the end consumer's country, SomnoMed shall assume responsibility for the warranty for material defects (but not for damage to the contractual merchandise by the end consumer).

Refer to [www.somnomed.com](http://www.somnomed.com) for the latest information on SomnoMed's warranty policy.

The warranty does not cover damage from improper use, abuse, pet chewing, exposure to excessive heat, exposure to caustic cleaners, or where dental work is completed post order placement.

## Returning your device

---

Warranty repair work can only be completed by an authorized SomnoMed facility. You must return your device through your dental sleep professional. Do not return your device directly to SomnoMed. Do not include any accessories with your return. New accessories will not be issued with device repairs.

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**SomnoMed Ltd.**

Level 3, 20 Clarke Street  
Crows Nest  
NSW 2065 Australia

**SomnoMed UK Limited**

Compass Point Business Park,  
Raleigh House,  
14c Stocks Bridge Way, St Ives  
PE27 5JL, United Kingdom

**SomnoMed AG**

Steinhauserstrasse 74  
6300 Zug, Switzerland

**SomnoMed France SAS**

Les Espaces Mermoz  
36 Impasse Louis Blériot  
44150 Ancenis, France

**SomnoMed Canada Inc.**

221 Talbot Street West  
Leamington  
ON N8H 1N8, Canada

**SomnoMed Inc.**

Plano , TX 75024  
United States of America

Visit [somnomed.com](https://www.somnomed.com)  
for worldwide locations.

Copyright © 2024 SomnoMed, All rights reserved.

SomnoMed, SomnoDent, Herbst Advance and SomnoDent Herbst Advance Elite  
are trademarks and/or registered trademarks of SomnoMed Ltd.

30ENFRES1-1003A (2025-03-05)